



POTASSIUM/K

Potassium is measured by ion-selective electrode potentiometry. In the calculation of results for potassium, concentration is related to potential through the Nernst equation.

The i-STAT System uses direct (undiluted) electrochemical methods. Values obtained by direct methods may differ from those obtained by indirect (diluted) methods.¹

See below for information on factors affecting results. Certain substances, such as drugs, may affect analyte levels *in vivo*.²

If results appear inconsistent with the clinical assessment, the patient sample should be retested using another cartridge.

Intended Use

The test for potassium, as part of the i-STAT System, is intended for use in the *in vitro* quantification of potassium in arterial, venous, or capillary whole blood.

Contents

Each i-STAT cartridge contains one reference electrode (when potentiometric sensors are included in the cartridge configuration), sensors for the measurement of specific analytes and a buffered aqueous calibrant solution that contains known concentrations of analytes and preservatives. For cartridges that contain a sensor for the measurement of potassium, a list of reactive ingredients is indicated below:

Reactive Ingredient
Potassium (K ⁺)

Metrological Traceability

The i-STAT System test for potassium measures potassium amount-of-substance concentration in the plasma fraction of arterial, venous, or capillary whole blood (dimension mmol L⁻¹) for *in vitro* diagnostic use. Potassium values assigned to i-STAT's controls and calibration verification materials are traceable to the U.S. National Institute of Standards and Technology (NIST) standard reference material SRM956. i-STAT System controls and calibration verification materials are validated for use only with the i-STAT System and assigned values may not be commutable with other methods. Further information regarding metrological traceability is available from Abbott Point of Care Inc..

Expected Values

Test/Abbreviation	Units*	Reportable Range	Reference Range ³
Potassium/K	mmol/L(mEq/L)	2 – 9	3.5 – 4.9**

*The i-STAT System can be configured with the preferred units.

**The reference range for potassium listed above has been reduced by 0.2 mmol/L from the range cited in Reference 3 to account for the difference between serum and plasma results.

The i-STAT reference range for whole blood listed above is similar to reference ranges derived from serum or plasma measurements with standard laboratory methods.

The reference range programmed into the analyzer and shown above is intended to be used as a guide for the interpretation of results. Since reference ranges may vary with demographic factors such as age, gender and heritage, it is recommended that reference ranges be determined for the population being tested.

Clinical Significance

Tests for potassium in the blood are important in the diagnosis and treatment of patients suffering from hypertension, renal failure or impairment, cardiac distress, disorientation, dehydration, nausea and diarrhea. Some causes of increased values for potassium include renal glomerular disease, adrenocortical insufficiency, diabetic ketacidosis (DKA), sepsis and in vitro hemolysis. Some causes of decreased values for potassium include renal tubular disease, hyperaldosteronism, treatment of DKA, hyperinsulinism, metabolic alkalosis and diuretic therapy.

Performance Characteristics

The typical performance data summarized below was collected in health care facilities by health care professionals trained in the use of the i-STAT System and comparative methods.

Precision data were collected in multiple sites as follows: Duplicates of each control fluid were tested in the morning and in the afternoon on five days for a total of 20 replicates. The averaged statistics are presented below.

Method comparison data were collected using CLSI guideline EP9-A⁴. Venous blood samples were collected in lithium heparin Vacutainer[®] tubes and analyzed in duplicate on the i-STAT System. A portion of the specimen was centrifuged and the separated plasma was analyzed in duplicate on comparative methods within 20 minutes of collection.

Deming regression analysis⁵ was performed on the first replicate of each sample. In the method comparison table, n is the number of specimens in the data set, Sxx and Syy refer to estimates of imprecision based on the duplicates of the comparative and the i-STAT methods respectively, Sy.x is the standard error of the estimate, and r is the correlation coefficient.*

Method comparisons will vary from site to site due to differences in sample handling, comparative method calibration and other site specific variables.

Interference studies were based on CLSI guideline EP7.⁶

*The usual warning relating to the use of regression analysis is summarized here as a reminder: For any analyte, "if the data is collected over a narrow range, the estimate of the regression parameters are relatively imprecise and may be biased. Therefore, predictions made from these estimates may be invalid".⁴ The correlation coefficient, r, can be used as a guide to assess the adequacy of the comparative method range in overcoming this problem. As a guide, the range of data can be considered adequate if $r > 0.975$.

Precision Data (mmol/L or mEq/L)

Aqueous Control	Mean	SD	%CV
Level 1	2.85	0.038	1.3
Level 3	6.30	0.039	0.6

Method Comparison (mmol/L or mEq/L)

	Beckman Synchron CX®3	Kodak Ektachem™ 700	Nova STAT Profile® 5
n	189	142	192
Sxx	0.060	0.031	0.065
Syy	0.055	0.059	0.055
Slope	0.97	1.06	0.99
Int't	0.02	-0.15	-0.01
Sy.x	0.076	0.060	0.112
Xmin	2.8	3.0	2.8
Xmax	5.7	9.2	5.8
r	0.978	0.993	0.948

Cartridge Comparison

The performance characteristics of the sensors are equivalent in all cartridge configurations. System difference analysis was performed on 40 patient samples using the i-STAT 6+ and i-STAT EC4+ cartridges. In the 3.0–5.0 mmol/L range the average difference was 0.049.

Factors Affecting Results*

If heparinized whole blood is allowed to stand before testing, potassium values will first decrease slightly, then increase over time. Potassium values will increase in iced specimens.

Potassium values from anticoagulated samples are preferred to serum values because 0.1 to 0.7 mmol/L potassium can be released from platelets¹ and red blood cells during the clotting process. Potassium values obtained from skin puncture samples may vary due to hemolysis or an increase in tissue fluid from improper technique during the collection procedure.

*It is possible that other interfering substances may be encountered. These results are representative and your results may differ somewhat due to test-to-test variation. The degree of interference at concentrations other than those listed might not be predictable.

References

1. N.W. Tietz, E.L. Pruden, O. Siggaard-Andersen, "Electrolytes " in Tietz Textbook of Clinical Chemistry—Second Edition, C.A. Burtis and E.R. Ashwood, eds. (Philadelphia: W.B. Saunders Company, 1994).
2. D.S. Young, Effects of Drugs on Clinical Laboratory Tests, 3rd ed. (Washington, DC: American Association of Clinical Chemistry, 1990).
3. B.E. Statland, Clinical Decision Levels for Lab Tests (Oradell, NJ: Medical Economic Books, 1987).
4. CLSI. *Method Comparison and Bias Estimation Using Patient Samples; Approved Guideline*. CLSI document EP9-A [ISBN 1-56238-283-7]. CLSI, 940 West Valley Road, Suite 1400, Wayne, Pennsylvania 19087-1898, USA 1995.
5. P.J. Cornbleet and N. Gochman, "Incorrect Least-Squares Regression Coefficients in Method-Comparison Analysis," *Clinical Chemistry* 25:3, 432 (1979).
6. CLSI. *Interference Testing in Clinical Chemistry; Proposed Guideline*. CLSI document EP7-P [ISBN 1-56238-020-6]. CLSI, 940 West Valley Road, Suite 1400, Wayne, Pennsylvania 19087-1898, USA 1986.

i-STAT is a registered trademark of Abbott Laboratories, East Windsor, NJ USA. Vacutainer is a registered trademark of Becton Dickinson and Company, Franklin Lakes, NJ USA. CX 3 is a registered trademark of Beckman Coulter Incorporated, Fullerton, CA USA. Ektachem was a trademark of Kodak Clinical Diagnostics. This system is now the Vitros distributed by Ortho-Clinical Diagnostics, Rochester, NY USA. Stat Profile is a registered trademark of Nova Biomedical, Waltham, MA USA.



Abbott Point of Care Inc.
Abbott Park, IL 60064 • USA



Emergo Europe
P.O. Box 18510
2502 EM The Hague
The Netherlands
Tel: (31)70 345 8570
Fax: (31)70 346 7299



©2008 Abbott Point of Care Inc.. All rights reserved. Printed in USA.